

ISOL/8

Instruction Manual

Version: 002

Product name: ISOL/8 Short Sleeved Patient Examination Gown
Models:

Product numbers: IS8PG01
IS8PG02

Manufacturer: ISOL8 Healthcare Ltd
Unit 4 Joyce House
Barrack Square
Ballincollig
Co. Cork
Ireland
P31 HW35

Product description: Gowns worn by patients during medical examinations, procedures and inpatient stays.

Size(s): L (Fits sizes S-XL), XXL (Fits sizes XXL-3XL)

Fabric: SMS 40g/m² (Spunbond-Meltblown-Spunbond) Polypropylene

Intended use:

This short-sleeved gown is intended for use by patients in order to provide dignified cover and facilitate easy examination of the patient by healthcare professionals during examinations and surgical or diagnostic procedures.

How to wear:

- 1) Wash hands thoroughly before wearing gown.
- 2) Remove clothing to the level asked by your healthcare professional.
- 3) Place arms into the sleeves with the open part of the gown facing backwards.

- 4) Tie the neck ties in a bow or knot.
- 5) Tie the waist-level ties at the rear using a bow or knot ensuring that the back is adequately covered.
- 6) After use dispose of product in accordance with your organisation's guidelines.
- 7) Wash hands after use.

Warnings & Contraindications:

- 1) Do not use this product if you are allergic to polypropylene.
- 2) Do not use this product after the expiry date has passed.
- 3) Do not use this product if the packaging is opened, wet or visibly damaged.
- 4) Do not expose product to flames, heat or high intensity light source.
- 5) This product is only to be used under the supervision of trained medical personnel.
- 6) Keep packaging and products away from children.

Single-use only:

This device is a disposable single-use product. It is not to be cleaned, re-sterilised or reused under any circumstance.

Storage conditions:

Store in a dry, cool, ventilated environment away from flammable and corrosive materials and direct sunlight.

Transport conditions:

During transport, avoid trauma to the carton, direct sunlight, rain, snow and other forms of precipitation.

Disposal Instructions:

Please dispose of gown in accordance with your organisation's guidelines.

Recycling Instructions:

Carton: 100% recyclable (cardboard)

Product polythene bag: 100% recyclable (polythene)

Gown: Not to be recycled. Please dispose of in accordance with your organisation's guidelines.

Sterility:

Product is non-sterile.

Product labelling explanation:



Product is a medical device (MDR)



Product is free from latex materials



Product is for single-use only



Do not use if the package is damaged



Keep dry and away from moisture



Catalogue/ product number



Manufacturer information



Date of manufacture



LOT/Batch code



Product use-by date



CE Mark
MDR (EU)
2017/745



Product is non-sterile